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## Warnings and precautions

- *Not suitable for persons younger than 18 years.*
- *Use the products of the sample collection set exclusively like described in the instructions.*
- *If you suffer from a bleeding disorder or take anticoagulant medication, you are pregnant or breast feeding, please talk to your physician before using the blood collection set.*
- *Don't use the sample collection set after exceeding the best before date (last day of the month and year written on the packaging).*
- *Keep the alcohol pads away from fire and potential sources of ignition.*
- *Don't bring the alcohol pads in contact with your eyes or mucous membranes and don't breath in the released fumes.*
- *Check the cap of the safety lancet before use and don't use the lancet in case of damage.*
- *By no means should the products of the blood collection set be used more than once.*

- *Please don't touch the spots of the blood spot card where the blood should be applied.*
- *The dried blood spot card should not come into contact with any other liquids.*
- *Any serious incidents that occur should be reported to the procedure pack producer and the competent authority.*
- **Emergency contact information: In emergencies, you can reach the medical on-call service by dialing 116 117. In life-threatening situations, please dial the emergency number 112.**

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## Test activation

- Log in to your account and click on "Activate test kit".
- Select the appropriate test kit from the list.
- Scan the barcode on the test kit with your smartphone or your computer's camera.
- Alternatively, you can also enter the number below the barcode manually.

**Without activating the test kit, the samples sent in cannot be assigned to you. In this case a new test kit must be ordered at your own expense.**

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Prepare all materials for sampling and carry out the sample collections one after the other as shown. The swabs are placed in the yellow urin sample tube.

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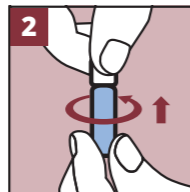
**Attention:** Wash hands with warm water and soap, rub hands and massage the finger with gentle pressure from palm to fingertip. Let arm hang after punctation.

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
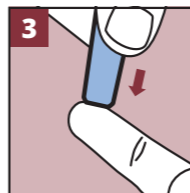
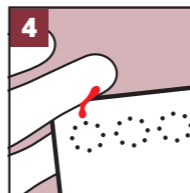
Apply the lancet at the side of the fingertip. Due to this pressure the lancet will automatically activate. Let arm hang after punctation. Each lancet should and can **only be used once**.

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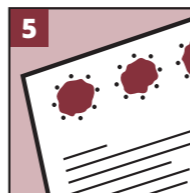
Wipe off the first drop of blood with the compress. Drip the following droplets on the preprinted circles on the dried blood spot card.

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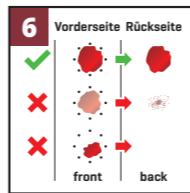**Fill all circles of the dried blood spot card.** Apply the blood on the front side. The bleeding can be treated with the included swab and plaster. If necessary, use the second lancet to puncture again.

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
The circles should be **sufficiently filled and soaked** (area  $\varnothing > 6$  mm per circle). The card should look approximately the same on both sides.

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Let the dried blood spot card dry for at least **2-4 hours** at room temperature by leaning the card against an object. Don't use a heating source for drying.

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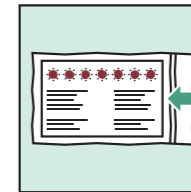Place the **dried blood spot card** together with the desiccant bag back into the foil pouch. Send the **foil pouch** in the large **return envelope**.

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## Intended use

The "dried blood sampling set" from Deutsche Aidshilfe / s.a.m Health is used as a set for taking and sending a dried blood sample for suitable and validated laboratory tests. It can be used by healthcare professionals as well as for self-testing. For *in vitro* diagnostic purposes only.